



Visual Nurse Call 4000 Single Status
*Recommended for the Latching Series Only



Emergency Station with Momentary Switch E-144



Description: The Momentary Emergency Station (E-144) to be used with a latching Annunciator panel features satin finished stainless steel faceplates with silk-screened designations. It may be flush mounted on standard outlet boxes with single-gang rings, or surface mounted on Wiremold 5748 or their equivalents, with a depth requirement of 1 3/4". Field wiring terminations are with pig-tails.

Operation: Pulling the four-foot nylon cord on the E-144 activates the *momentary switch* on the emergency station placing a call for help. Station is designed to be used with a latching Annunciator panel.

Mounting: The satin finish stainless steel faceplate with silk-screened designations may be flush mounted on a standard outlet box with single-gang plaster ring, or surface mounted on a Wiremold 5748 or equivalent.

Engineering Specifications: The contractor shall furnish and install the CORNELL E-144 Emergency Stations as indicated on the plans. Faceplates shall be satin-finish stainless steel with silk-screened designations. Stations shall be mounted on standard outlet boxes with single-gang rings or Wiremold 5748 or equivalent.

Technical Information:

- Power Requirements: 24VDC
- Operating Environment: 50-120°F Indoor Non-condensing
- Physical Faceplate Dimensions: 4 1/2 "H x 2 3/4 "W (1 Gang)
- Mounting: Single-Gang back box with 1 3/4" minimum depth
- Wiring: Terminations are Pigtailed

CORNELL COMMUNICATIONS, INC.

Milwaukee, Wisconsin USA • 800-558-8957 • 414-351-4660 • Fax 414-351-4657
www.cornell.com • sales@cornell.com





DE144C
Rev 171227

Available in 1-6 gang sizes:

*stations are only UL Listed in the 1 gang size

* For correct part number, change first number (1) to gang size needed

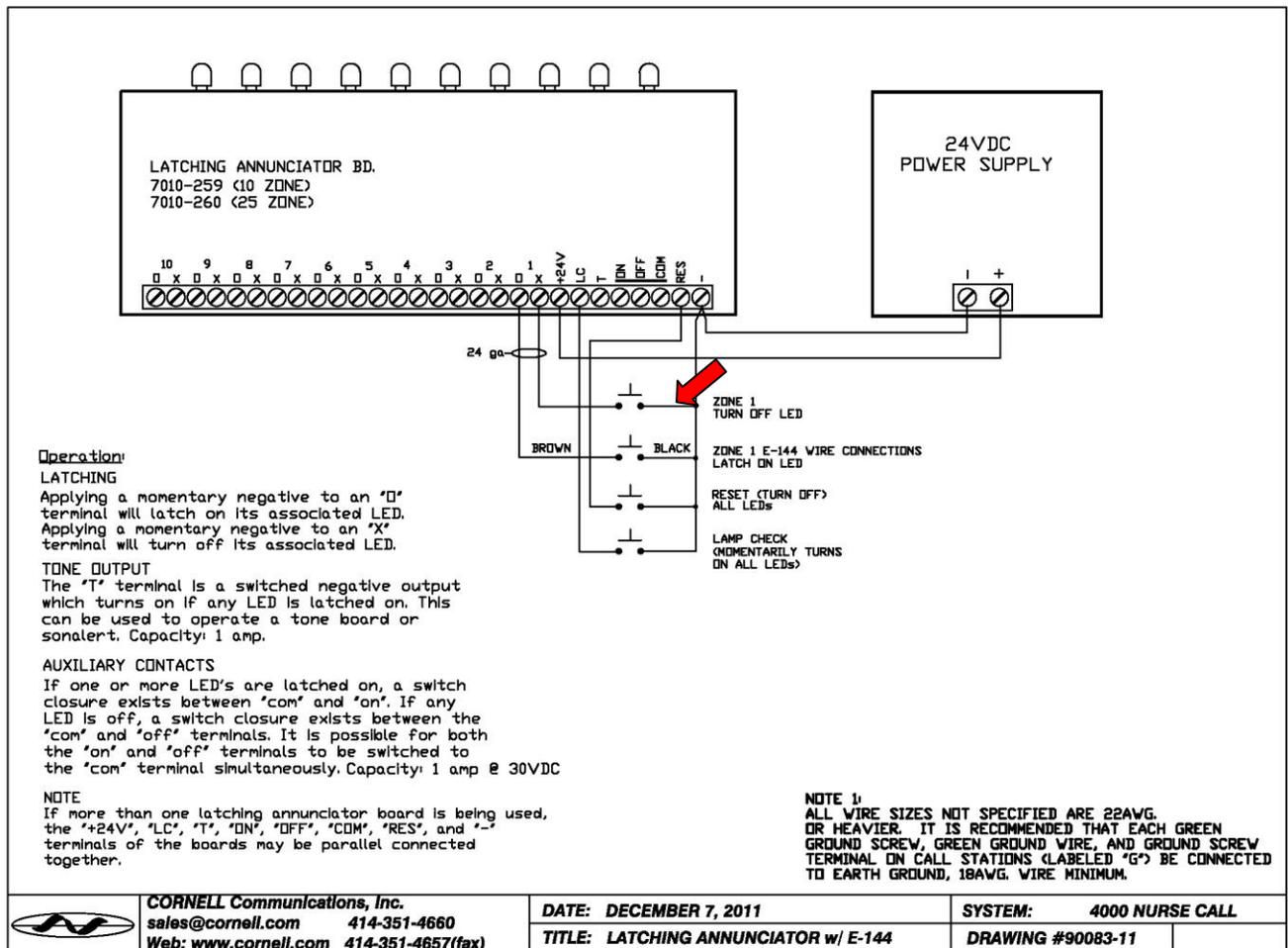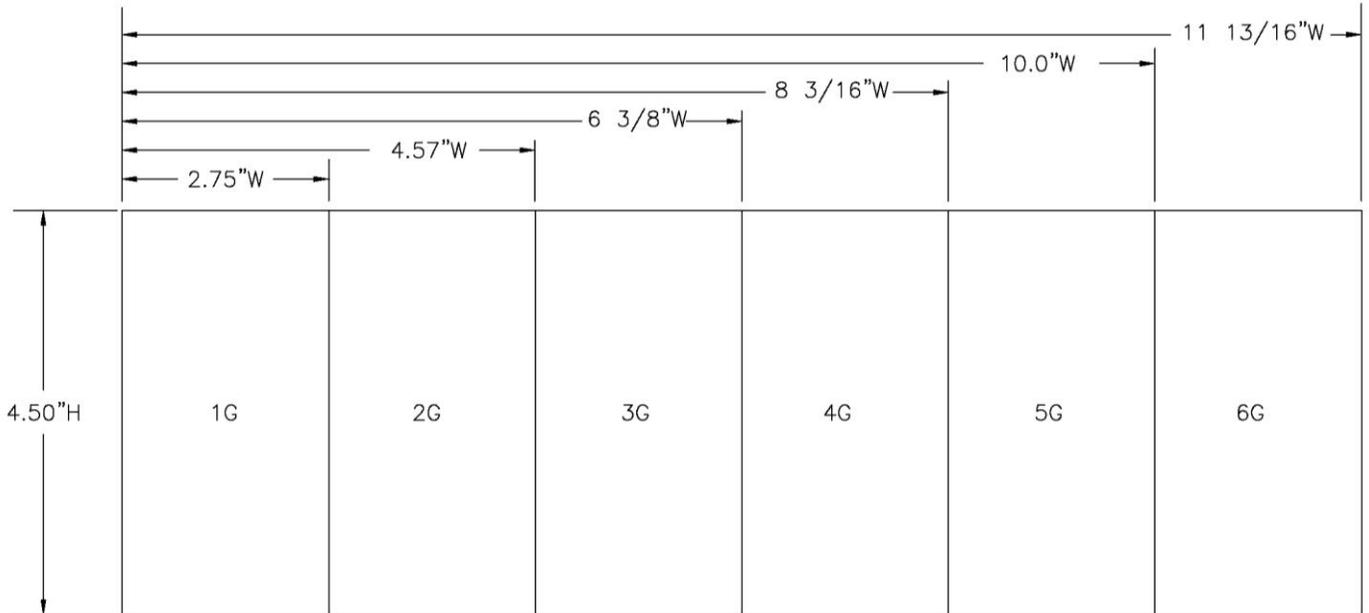

C