



- NOTES:**
- ANY SET OF 3 TERMINALS (A, B, OR C) MAY BE USED FOR ANY OF THE 3 NEGATIVE "FEEDS" (S, F, OR P). A, B, & C USE IDENTICAL CIRCUITS.
 "A" = TOP BULB, AS SHOWN
 "B" = MIDDLE BULB, AS SHOWN
 "C" = BOTTOM BULB, AS SHOWN
 WHEN PRIORITY "P" IS NOT USED, USING THE OUTER BULBS (A&C) MAY PROVIDE BETTER VIEWING ANGLES.
 - TERMINAL #10:**
 THIS TERMINAL PROVIDES A NEGATIVE OUTPUT, WHICH WILL MATCH THE SUMMED OUTPUT OF A+B+C



CORNELL Communications, Inc.
 sales@cornell.com 414-351-4660
 Web: www.cornell.com 414-351-4657(fax)

DATE: APRIL 1, 2008

TITLE: Z-103 WIRING

SYSTEM: 4000 NURSE CALL

DRAWING # 90008-08 **DZ103W**